

Atlantis® (IO) FLO-S compatibility chart

The scan body Atlantis (IO) FLO-S is used when scanning for Atlantis suprastructures. It can be used on implant/abutment level for all positions in the mouth of the patient or on the master model during scanning. Refer to instructions for use for further information, 2663-IFU Atlantis IO FLO-S.

This chart describes what implants/abutments the Atlantis (IO) FLO-S is compatible with and the content of Dentsply Sirona Implants Atlantis suprastructures (IO) FLO-S libraries. The libraries are used for digital design guidance for Atlantis suprastructures and for building and printing models. The libraries can be used for 3Shape and Exocad design software.

Note: The FLO-S scan body without the IO marking can only be used for model scanning and is not approved for intraoral scanning. The IO FLO-S scan body will be used for both intraoral and model scanning.

Each Atlantis (IO) FLO-S reference number is sold as a kit containing a tray with 6 (IO) FLO-S for storing the non-sterile (IO) FLO-S and a Tag (to be placed in the Atlantis FLO-S case (REF) 36701) for 1 specific connection.



Manufacturer	Implant system	FLO-S id	FLO-S Movex no (Kit)	Intraoral scanning	Library information				
					Available in library	Elos Accurate Analog for Printed Models	Elos Accurate Hybrid Base Non-Engaging	Elos Accurate Hybrid Base H Non-Engaging	
Dentsply Sirona	IMPLANT LEVEL								
	EV Connection: PrimeTaper EV Implant / OmniTaper EV Implant / Astra Tech Implant EV								
	EV Connection (XS) Green	A04K	68020018	YES	YES	PMA-ATE30-1 (68020083)			
	EV Connection (S) Purple	A04L	68020019	YES	YES	PMA-ATE36-1 (68020084)			
	EV Connection (M) Yellow	A04M	68020020	YES	YES	PMA-ATE42-1 (68020085)			
	EV Connection (L) Blue	A04N	68020021	YES	YES	PMA-ATE48-1 (68020086)			
	EV Connection (XL) Brown*	A04O	68020022	YES	YES	PMA-ATE54-1 (68020087)			
	Astra Tech Implant EV Profile								
	4.2 Yellow	A04M	68020020	YES	YES	PMA-ATE42P-1 (68020088)			
	4.8 Blue	A04N	68020021	YES	YES	PMA-ATE48P-1 (68020089)			
	Astra Tech Implant System TX								
	OsseoSpeed TX 3.0 – Yellow	A04C	68020023	YES	YES	PMA-ATO30-1 (68020090)			
	OsseoSpeed TX 3.5, 4.0 – Aqua	A04D	68020024	YES	YES	PMA-ATO35-1 (68020091)			
	OsseoSpeed TX 4.5, 5.0 – Lilac	A04E	68020025	YES	YES	PMA-ATO45-1 (68020092)			
	Xive / Frialit								
	Xive S 3.0	F01A	68020026	YES	YES	PMA-DXI30-1 (68020093)			
	Xive S 3.4 / Frialit 3.4	F01B	68020027	YES	YES	PMA-DXI34-1 (68020094)			
	Xive S 3.8 / Frialit 3.8	F01C	68020028	YES	YES	PMA-DXI38-1 (68020095)			
	Xive S 4.5 / Frialit 4.5	F01D	68020029	YES	YES	PMA-DXI45-1 (68020096)			
	Xive S 5.5 / Frialit 5.5	F01E	68020030	YES	YES	PMA-DXI55-1 (68020097)			

Please turn over →

* Note: Not applicable for PrimeTaper EV Implant.

All trademarks and company names are the property of their respective owners.

Elos Accurate® is a registered trademark.

Manufacturer	Implant system	FLO-S id	FLO-S Movex no (Kit)	Intraoral scanning	Library information			
					Available in library	Elos Accurate Analog for Printed Models	Elos Accurate Hybrid Base Non-Engaging	Elos Accurate Hybrid Base H Non-Engaging
Dentsply Sirona	ABUTMENT LEVEL							
	EV Connection: DS PrimeTaper Implant / DS OmniTaper Implant / Astra Tech Implant EV							
	MultiBase Abutment EV	A04S	68020034	YES	YES	PMA-MUA45-1 (68020099)	HBN-MUA45-1	HBN-MUA45H-1
	Uni Abutment EV	A04F	68020031	YES	YES	PMA-AUE33-1 (68020100)	HBN-AUE33-1	HBN-AUE33H-1
	Astra Tech Implant System TX							
	20° UniAbutment	A04A	68020032	YES	YES	PMA-ATU20-1 (68020101)	HBN-ATU20-1	HBN-ATU20H-1
	45° UniAbutment	A04B	68020033	YES	YES	PMA-ATU45-1 (68020102)	HBN-ATU45-1	
	Xive							
	Xive MP/TG 3.4,3.8	F01F	36745-6		YES			
	Xive MP/TG 4.5	F01G	36746-6		YES			
	Xive MP 5.5	F01H	36747-6		YES			
	Ankylos							
	Ankylos Balance Base Narrow D4.2	A03A	68020035	YES	YES	PMA-DBA40-1 (68020103)	HBN-DBA40-1	HBN-DBA40H-1
Ankylos Balance Base D5.5	A03B	36730-6		YES				
MIS	IMPLANT LEVEL							
	Internal Hex Narrow SEVEN – M4	M04D	68020043	YES	YES			
	Internal Hex Standard SEVEN – M4	M04A	68020044	YES	YES			
	Internal Hex Wide SEVEN – M4	M04B	68020045	YES	YES			
	Conical Connection Narrow C1	M04E	68020046	YES	YES			
	Conical Connection Standard C1 – V3	M04F	68020047	YES	YES			
	Conical Connection Narrow V3	M04E	68020046	YES	YES			
	Conical Connection Wide C1	M04G	68020048	YES	YES			
	ABUTMENT LEVEL							
	Multi-Unit Abutment	M04H	68020049	YES	YES			
BioHorizons	IMPLANT LEVEL							
	Internal/ Tapered 3.5	B03A	36710-6					
	Internal/ Tapered 4.5	B03B	36711-6					
	Internal/ Tapered 5.7	B03C	36712-6					
	ABUTMENT LEVEL							
Multi-unit abutment	A04S	68020034						

Please turn over 

Manufacturer	Implant system	FLO-S id	FLO-S Movex no (Kit)	Intraoral scanning	Library information			
					Available in library	Elos Accurate Analog for Printed Models	Elos Accurate Hybrid Base Non-Engaging	Elos Accurate Hybrid Base H Non-Engaging
Biomet 3i	IMPLANT LEVEL							
	Certain 3.25, 4/3	301F	36716-6					
	Certain 4.0, 5/4	301H	36717-6					
	Certain 5.0	301J	36718-6					
	Certain 6.0	301L	36719-6					
	Certain Prevail 3/4/3, 4/3	301F	36716-6					
	Certain Prevail 4/5/4, 5/4	301H	36717-6					
	Certain Prevail 5/6/5, 6/5	301J	36718-6					
	Certain XP4/5	301J	36718-6					
	Certain XP 5/6	301L	36719-6					
Camlog	ABUTMENT LEVEL							
	Low Profile Abutment	301E	36720-6					
	IMPLANT LEVEL							
	Screw-line Implant 3.3	C01E	36725-6					
	Screw-line/Root-line Implant 3.8	C01G	36726-6					
	Screw-line/Root-line Implant 4.3	C01H	36727-6					
Keystone Dental	Screw-line/Root-line Implant 5.0	C01I	36728-6					
	Screw-line/Root-line Implant 6.0	C01K	36729-6					
	IMPLANT LEVEL							
	PrimaConnex SD 3.3,3.5	K01A	36755-6					
	PrimaConnex RD 4.0,4.1	K01B	36754-6					
	PrimaConnex WD 5.0	K01C	36756-6					
Nobel Biocare	Genesis 3.8	K01A	36755-6					
	Genesis 4.5	K01B	36754-6					
	Genesis 5.5, 6.5	K01C	36756-6					
	IMPLANT LEVEL							
	NobelActive NP 3.5	N01A	68020050	YES	YES	PMA-NBA35-1 (68020119)		
	NobelActive RP 4.3, 5.0	N01B	68020051	YES	YES	PMA-NBA43-1 (68020120)		
	NobelActive WP 5.5	N01O	68020052	YES	YES	PMA-NBA60-1 (68020121)		
	NobelReplace NP 3.5	N01J	68020053	YES	YES	PMA-NBR35-1 (68020122)		
	NobelReplace RP 4.3	N01K	68020054	YES	YES	PMA-NBR43-1 (68020123)		
	NobelReplace WP 5.0	N01L	68020055	YES	YES	PMA-NBR50-1 (68020124)		
NobelReplace 6.0	N01M	68020056	YES	YES	PMA-NBR60-1 (68020125)			
Nobel Biocare	Branemark System NP 3.3	N01E	68020057	YES	YES	PMA-BRA35-1 (68020126)		
	Branemark System RP 3.75, 4.0	N01F	68020009	YES	YES	PMA-BRA41-1 (68020127)		
	ABUTMENT LEVEL							
	Multi-Unit Abutment RP	N01H	68020058	YES	YES	PMA-MUA45-1 (68020099)		

Please turn over 

All trademarks and company names are the property of their respective owners.

Elos Accurate® is a registered trademark.

Manufacturer	Implant system	FLO-S id	FLO-S Movex no (Kit)	Intraoral scanning	Library information			
					Available in library	Elos Accurate Analog for Printed Models	Elos Accurate Hybrid Base Non-Engaging	Elos Accurate Hybrid Base H Non-Engaging
Straumann	IMPLANT LEVEL							
	Bone Level 3.3 NC	S05G	68020059	YES	YES	PMA-SBO33-1 (68020129)		
	Bone Level 4.1, 4.8 RC	S05H	68020060	YES	YES	PMA-SBO41-1 (68020130)		
	Standard/Standard Plus 4.8 RN (Regular Neck)	S05D	68020061	YES	YES	PMA-SSY48-1 (68020131)		
	Standard/Standard Plus 4.8 WN (Wide Neck)	S05F	68020062	YES	YES	PMA-SSY65-1 (68020132)		
	ABUTMENT LEVEL							
	RN Abutment Level	S05C	36772-6			YES		
	WN Abutment Level	S05E	36777-6			YES		
	Bone Level Multi-Base Abutment D3.5	S05I	36768-6			YES		
	Bone Level Multi-Base Abutment D4.5	S05J	36769-6			YES		
	Bone Level Multi-Base Angled Abutment	S05K	36770-6			YES		
	Screw-Retained Abutment 3.5	S05O	68020069	YES	YES	PMA-SSA35-1 (68020135)		
	Screw-Retained Abutment 4.6	S05P	68020070	YES	YES	PMA-SSA46-1 (68020136)		
Zimmer Dental	IMPLANT LEVEL							
	Tapered Screw-Vent™ 3.5 / Screw Vent™ 3.3, 3.7	B03A	36710-6					
	Tapered Screw-Vent™ 4.5 / Screw Vent™ 4.5	B03B	36711-6					
	Tapered Screw-Vent™ 5.7	B03C	36712-6					
	ABUTMENT LEVEL							
	Tapered Abutment	Z01E	36779-6					

All trademarks and company names are the property of their respective owners.
Elos Accurate® is a registered trademark.

Tool for Elos Accurate® Analog for Printed Models	REF
Elos Accurate Analog Pliers	PMA-AIT-1 (68020141)
Elos Accurate Analog Insertion Screw	PMA-AIS-5 (68020142)
Elos Accurate Analog Insertion Pin	PMA-AIP-2 (68020143)

